BELSANA

engel

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Dear users,

We are pleased that you have chosen a product from BELSANA and thank you for the trust you have put in us. Please take a moment to read the product information carefully.

Intended use

The robust donning aid makes it easier for you to put on compression stockings independently, even if your mobility is limited, because you can put them on without bending over.

⚠ Important information

The donning aid must be issued by trained medical personnel. Your specialist retailer will also advise you on how to use your donning aid correctly. We test our products as part of a comprehensive quality assurance process. If you have any complaints, please contact your retailer. Do not attempt to repair any damage yourself, but return the product to the retailer you purchased it from. Please do not use damaged products or products no longer in proper condition to avoid injury and/or damage to the stockings.

Care instructions

The product is suitable for immediate use. Please clean the donning aid as needed with a damp cloth and, if necessary, with a mild detergent. Please check the material compatibility of the detergent on a small area of the donning aid before using it. Store the donning aid in a dry place and protect it from the sun and heat, if possible in its original packaging.

Material composition:

Iron wire, Epoxy polyester paint,
End caps: PVC

⚠ Risks and side effects

If you are sensitive to the material of the donning aid, you may experience skin irritation or allergic reactions during use. Paint damage to the donning aid, e.g. due to wear and tear, incorrect storage and the development of rust, or use of unsuitable detergents may also cause skin reactions. The donning aid must not come into contact with injured skin. We recommend the use of special gloves, e.g. BELSANA gripStar. These also help protect the compression stockings.

Warranty

We guarantee the functionality of the donning aid for 5 years. Once this period has expired, the products are excluded from the warranty. In general, defects which are caused by normal wear or improper use are excluded. Improper use also includes any subsequent change to the product which was not done by the manufacturer. The product is intended for the care of a single patient. If it is passed on to other persons for reuse, the manufacturer's product liability shall lapse.

Disposal

Please dispose of the product in accordance with local regulations after the end of its service life.

Reporting obligation

Due to legal regulations, patients and users are obliged to immediately report any serious incidents that have occurred in connection with the product which have led, or could have led, to a serious deterioration in health or death of the user to the manufacturer and the competent national authority (in Germany BfArM).

Place the donning aid in front of you so that the closed side is facing you and the handles are on top. Then put the stocking over the semi-circular frame's stirrup, folding the edge of the stocking outwards. Firmly pull the stocking over the frame until the inside of the heel is visible. The heel should be facing you.

Then step your foot into the frame until your toes have reached the tip of the stocking and the heel is also correctly positioned on the foot. Guide the donning aid upwards until you can comfortably reach the stocking with your hands.

⚠ Make sure that the donning aid is in a secure position to avoid slipping when putting on the stocking. Please take care not to trap your hands between the frame's stirrup and the moving handles. The optionally available BELSANA gripStar gloves not only protect the knitted fabric and make it easier to put on the stocking, but also minimise the risk of injury.

⚠ Please note:

The donning aid BELSANA engel (standard version) is suitable for a calf circumference up to max. 44 cm. For larger calf circumferences up to max. 50 cm we recommend the large version of the BELSANA engel.

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Zweigniederlassung der Ofa Bamberg GmbH
Ofa Bamberg GmbH