
BELSANA

Beine in Bestform

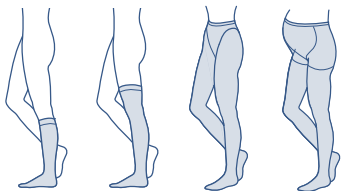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Dear User,

BELSANA support stockings are the result of decades of experience in the development of medical compression stockings. You have decided to choose a high quality health product. In addition to the medical aspects, we pay particular attention to the wearing comfort. BELSANA stockings feel pleasant, are comfortable to wear and attractive in appearance. Healthy living made easy. Whether in a standing or sitting position, when travelling, participating in sports or simply because you wish to do yourself some good in everyday life.

⚠ Important information

The stockings must be obtained from a medically trained specialist retailer. Your pharmacy or specialist retailer will also advise you on how to correctly put on your stockings. If you experience leg pain or skin irritation while wearing these stockings, please consult your doctor immediately. You can prevent skin reactions in the area around the grip tops by slightly adjusting the grip tops several times during the day. Please contact your doctor about wearing the stockings in combination with using other medical devices. We test our products within the scope of a comprehensive quality assurance. However, if you still have cause for complaint, please contact your pharmacy or specialist retailer. Do not attempt to perform repairs yourself. Return the stockings to the pharmacy or retailer who sold them to you. Any pilling of the material that may arise due to friction with other garments will not reduce the stocking's quality.

Warranty

The lifetime of support stockings is limited to 5,5 years when stored properly. The lifetime of support stockings with skin-caring ingredients is limited to 3,5 years when stored properly. With regular use and proper care, we guarantee optimal compressive effect for 6 months. After this period, the products are excluded from the warranty. Defects attributable to normal wear and tear or improper use are not covered by the guarantee. Improper use also includes any subsequent modification to the product that has not been carried out by the manufacturer. Slight colour deviations, which may arise due to the individual dyeing process, are not considered to be a defect. Corresponding goods are therefore excluded from exchanges and returns. The product is intended for single use only. If it is passed on to other persons for reuse, the manufacturer's product liability is no longer valid.

Indications/Intended use

Feeling of heaviness and tiredness in the legs. Prophylactic effect against vein complaints and deep vein thrombosis caused by prolonged travel in people with healthy veins. Consult your doctor if in doubt as to whether there is any indication of venous disease that requires treatment.

Contraindications

- advanced peripheral arterial occlusive disease
- congestive heart failure
- septic phlebitis
- Phlegmasia cerulea dolens

Risks to consider

- marked weeping dermatoses
- allergies to support stocking materials
- allergies to support stocking ingredients
- severe sensitivity disorders of the extremity
- advanced peripheral neuropathy (e.g. diabetes mellitus)
- primary chronic polyarthritis

If you have any questions, please ask your doctor.

Effective principle of support stockings

The pressure decreases from the bottom to the top and promotes blood circulation and supports the venous system.

⚠ Risks and side effects

Users with sensitive skin may experience itching and skin irritation when using support stockings. Adequate skin care is therefore recommended. If you suffer the following symptoms, please remove the stockings immediately and consult your doctor before continuing using them: blue or white coloration of the toes, sensations of discomfort and numbness, increasing pain, shortness of breath and sweating, acute movement restrictions.

Mandatory reporting

Due to legal regulations within the EU, patients and users are obligated to report any serious events with the use of a medical device to both the manufacturer and the competent national authority (in Germany BfArM) without any delay.

Disposal

After the end of the usage period, please dispose of the product in accordance with local regulations.

Care instructions

The product is suitable for immediate use. We recommend washing the product before wearing it for the first time. Then wash daily by hand or on a delicate fabric program. Wash the stockings separately with mild detergent without fabric softener (e.g., BELSANA cleanStar40 detergent) at a maximum of 40°C and rinse well. The stockings can be spun gently, but do not wring them out. Dry your stockings in air or in a dryer on a gentle cycle. Do not place them on a heater or in direct sunlight. Never iron the stockings.

To maintain the effectiveness of your stockings, please do not use chemical cleaning agents, petrol, or fabric softeners. These substances damage the knitted fabric. Support stockings are largely resistant to fats and oils, but should not come into contact with ointments. Stockings with silicone-coated grip tops should generally not come into contact with solvents, ointments and re-greasing washing lotions and body creams. Otherwise, the adhesion of the grip tops could be permanently impaired. Store the stockings in a dry place away from sun and heat, in the original packaging if possible.

Please wash inside out and do not use any fabric softener!

 Wash on a delicate or gentle cycle

 Do not bleach

 Dry gently and carefully

 Do not iron

 No chemical dry cleaning

Tips for putting the stockings on

Carefully pull your stockings/pantyhose up your leg from the bottom to the top, do not pull them up from the top edge. Please avoid any damage that could be made by jewellery or long fingernails.

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Zweigniederlassung der Ofa Bamberg GmbH
Ofa Bamberg GmbH