# BELSANA

Beine in Bestform

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**Dear User,**

We are delighted that you have chosen BELSANA compression stockings and thank you for putting your trust in us. Please take a moment to read through this product information carefully. For optimal compression therapy effectiveness!

**Intended use**

Compression stockings are used to treat phlebological and lymphological arm and leg conditions as well as lipedema. Different products are used depending on the indication.

**Effect of medical compression stockings**

Products tailored to the respective indication and controlled pressure accelerate blood reflux, improve microcirculation in the tissue and increase lymph flow.

**Care instructions**

The product is suitable for immediate use. We recommend washing the product before wearing it for the first time. Then wash daily by hand or on a delicate fabric program. Wash the stockings separately with mild detergent without fabric softener (e.g., BELSANA cleanStar40 detergent) at a maximum of 40 °C and rinse well. The stockings can be spun gently, but do not wring them out. Dry

your compression stockings in air or in a dryer on a gentle cycle. Do not place them on a heater or in direct sunlight. Never iron the stockings. To maintain the effectiveness of your stockings, please do not use chemical cleaning agents, petrol, or fabric softeners. These substances damage the knitted fabric. Compression stockings are largely resistant to fats and oils, but should not come into contact with ointments. Stockings with

silicone-coated grip tops should generally not come into contact with solvents, ointments, re-greasing washing lotions, and body creams. Otherwise, the adhesive strength of the grip tops could be permanently impaired (leg hair and dry skin can also limit the function of the grip tops). Store the stockings in a dry place away from sun and heat, in the original packaging if possible.

**IMPORTANT INFORMATION**

Only wear compression stockings after consultation with your doctor. Wearing your stockings daily and holding regular consultations with your doctor will ensure successful treatment! The stockings must be obtained from a medically trained specialist retailer. Your pharmacy or specialist retailer will also advise you on how to put the compression stockings on correctly. If you experience leg pain or skin irritation while wearing these stockings, please consult your doctor immediately. You can prevent skin reactions in the area around the grip tops

by slightly adjusting the grip tops several times during the day. We test our products within the scope of a comprehensive quality assurance. However, if you still have cause for complaint, please contact your pharmacy or specialist retailer. Do not attempt to perform repairs yourself. Return stockings to the pharmacy or specialist retailer who sold them to you. Any pilling of the material that may arise due to friction with other garments will not reduce the stocking's quality. Please contact your doctor about wearing the stockings in combination with using other medical devices.

**Please wash the right way round and do not use any fabric softener!**

-  Wash on a delicate or gentle cycle
-  Do not bleach
-  Dry gently and carefully
-  Do not iron
-  No chemical dry cleaning

**Indications (see also AWMF Medical Compression Therapy Guideline 037/005, Version 12/18)**

- Improvement of venous symptoms and quality of life with chronic venous diseases
- Prevention and treatment of venous edema
- Prevention and treatment of venous skin alterations
- Eczema and pigmentation
- Lipodermatosclerosis and atrophie blanche
- Therapy and pain reduction for venous leg ulcers, including relapse prevention
- Treatment of arterial and venous leg ulcers (see contraindications!)
- Varicose veins
- Initial phase after treatment for varicose veins
- Functional venous insufficiency
- Venous malformations
- Venous thrombosis
- Condition after thrombosis
- Post-thrombotic syndrome
- Thrombosis prophylaxis in mobile patients
- Lymphedema
- Oedema in pregnancy
- Post-traumatic/post-operative edema
- Cyclic idiopathic edema
- Stage II lipedema and above
- Venous stasis resulting from immobility (arthrogenic congestive syndrome, paresis, and partial paresis of the extremity)
- Occupation-related edema (occupations involving long periods of standing and sitting)
- Medication-related edema, if change of medication not possible
- Obesity with functional venous insufficiency

- Inflammatory dermatoses of the legs
- Nausea, dizziness, and congestion complaints during pregnancy
- Condition after burns
- Scar treatment

**Contraindications**  
**Absolute contraindications:**

- Advanced peripheral artery disease (if any of the following values apply: ABPI < 0.5, ankle arterial pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO2 < 20 mmHg on instep)
  - Congestive heart failure
  - Septic phlebitis
  - Phlegmasia coerulea dolens
- Risks to be considered (therapy decision to be made based on benefit/risk assessment)**
- Weeping dermatoses
  - Allergies to compression stocking materials
  - Allergies to compression stocking ingredients
  - Severe sensitivity disorders of the extremities
  - Advanced peripheral neuropathy (e.g., diabetes mellitus)
  - Chronic polyarthritis

**No liability is accepted if these contraindications are not observed.**

**⚠ Risks and side effects**

Medical compression stockings can cause skin necrosis and pressure damage to the peripheral nerves. Users with sensitive skin may experience itching and skin irritation when using compression stockings. Adequate skin care is therefore recommended. If you suffer the following symptoms, please remove the stockings immediately and dis-

cuss further treatment with your doctor: blue or white coloration of the toes, sensations of discomfort and numbness, increasing pain, shortness of breath and sweating, acute movement restrictions.

**Warranty**

The shelf life of medical compression stockings is limited to 3 years when stored properly. Added to this is the wearing time, which is a maximum of 6 months with regular use and proper care. After this period, the products are excluded from the warranty. Defects caused by normal wear and tear or improper use are excluded from the warranty. Improper use also includes any subsequent modification of the product not carried out by the manufacturer. The product is intended for use by a single patient. Passing the product on to another person for secondary use renders the manufacturer's product liability null and void.

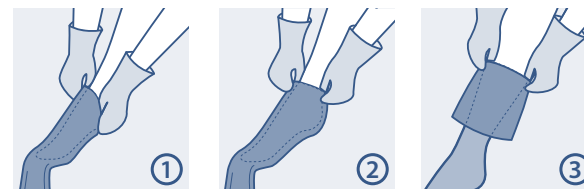
**Disposal**

After the end of the usage period, please dispose of the product in accordance with local regulations.

**Mandatory reporting**

Due to legal requirements, patients and users are obliged to report immediately any serious incidents which have occurred in connection with the product and which have resulted in, or could have resulted in, significant harm to health or even death to the manufacturer and the competent national authority (BfArM [Bundesinstitut für Arzneimittel und Medizinprodukte] (Federal Institute for Drugs and Medical Devices) in Germany).

**How to put them on, closed toe**

1. To put the stockings on, turn it inside out as far as the foot. Slip the foot section over your toes and then over the instep.
2. Now carefully pull the leg section over your heel. Please avoid pulling up the stockings at the end or grip top.
3. Pull the rest of the stocking up evenly without stretching it. Finally, adjust the stockings on the leg until it there are no wrinkles.

**How to put them on, open toe**

1. Putting on your open toe compression stockings is much easier with the supplied BELSANA dressing aid. Slip your foot into the dressing aid so that it reaches just above the heel.
2. Now pull the stockings on until the heel is in the right place. This is very easy with the BELSANA dressing aid.
3. To take out the dressing aid, pull back the forefoot section of the stockings.
4. Pull the dressing aid out over the tip of your toes.
5. Then carefully roll back the forefoot section of the stocking again. Then pull the rest of the stocking up evenly without stretching it. Finally, adjust the stocking on the leg until it there are no wrinkles.

**Material composition:**

Polyamide (Polyamid), elastane (Elastan) and, for some products, cotton (Baumwolle) or Merino wool (Merinowolle) are used.

Please refer to the textile label sewn into your stocking for the precise details.

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Zweigniederlassung der Ofa Bamberg GmbH
Ofa Bamberg GmbH